

CryoPen[®]



Reinventing Cryosurgery





LEADING-EDGE CRYOSURGERY

CryoPen is a range of innovative, powerful, hand-held cryosurgery tools, designed to completely remove all benign skin lesions quickly and cost-effectively. Treatment is very fast and can be easily integrated into daily practice.

CryoPen emits a microjet of nitrous oxide (N₂O) under high pressure (50-55 bars), achieving in-tissue temperature of -89°C for complete tissue necrosis in seconds.

No pain, no fuss, no aftercare

CryoPen treatment causes no pain, no damage to healthy tissue and little or no scarring. And it requires minimal post-treatment care.

Easy to use

If you have no experience performing cryosurgery, all you need are your diagnostic skills and a steady hand.

If you are experienced in using traditional cryosurgery tools, you'll be delighted by CryoPen's exceptional performance, millimeter precision and ease of use!



SEE



THE CRYOPEN IN ACTION !

Treat all benign skin lesions

Treat common pathologies
in just seconds:

Acrochordon, actinic keratosis and
cheilitis, capillary haemangioma,
cutaneous leishmaniasis, discoid
lupus erythematosus, filiform, flat,
vulgar, plantar and anogenital warts,
rosacea, lichen sclerosus, granuloma
annulare, molluscum contagiosum,
seborrheic keratosis and viral lesions.

Easy to manage

Each CryoPen device is supplied
with its own range of applicators.
Additional cartridges can be ordered.

Cartridges have a 2-year
shelf life and can be conveniently
stored at room temperature.



PERFECT REMOVAL OF SKIN LESIONS

CryoPen delivers exceptional results for the complete removal of all benign skin lesions in just one treatment session.



Acrochordon

Treatment:
10-15 sec.



Skin tag

Treatment:
5-10 sec.



Pigmented spot

Treatment:
3-5 sec.



Solar lentigo

Treatment:
3-5 sec.



Molluscum contagiosum

Treatment:
10-15 sec.



Hemangioma

Treatment:
5-10 sec.



Seborrheic keratosis

Treatment:
5-10 sec.



Wart

Treatment:
10-15 sec.



CHOOSE THE CRYOPEN THAT'S RIGHT FOR YOU

The CryoPen series includes a range of pen-like cryosurgery devices, applicators and cartridges to meet every practitioner's needs.

Your choice of device depends on your usage.

We will be happy to advise you.



CryoPen® M

Suitable for less-frequent use or for smaller/superficial lesions.
Takes 8g N₂O cartridges.

- 1-3mm Applicator |
- 2-6mm Applicator ||

CryoPen® X+

Suitable for intensive and varied use.
Available in 2 sizes, for 8g or 16g N₂O cartridges.

- 1-3mm Applicator |
- 2-6mm Applicator ||
- 4-8mm Applicator |||
- 7-20mm Applicator ||||
- 1-3mm long Applicator |
- 2-6mm long Applicator ||

CryoPen® B+

Suitable for frequent use or for larger/deeper lesions.
Takes 23.5g N₂O cartridges.

- 1-4mm Applicator |
- 2-6mm Applicator ||
- 4-8mm Applicator |||



Cartridges

8g N₂O

1 box contain 24 cartridges,
1 carton contain 12 boxes.

16g N₂O

1 box contain 6 cartridges,
1 carton contain 12 boxes.



CryoPen® o+

Suitable as an all-purpose cryosurgery device for regular use.

Available in 2 sizes, for 8g or 16g N₂O cartridges.

- 1-3mm Applicator |
- 2-6mm Applicator ||
- 4-8mm Applicator |||
- 7-20mm Applicator ||||

CryoPen® XP

Suitable for intensive and varied use.

Available in 2 sizes, for 8g or 16g N₂O cartridges.

- 1-3mm Applicator |
- 2-4mm Applicator ||
- 3-5mm Applicator |||
- 5-8mm Applicator ||||

23.5g N₂O

1 box contain 6 cartridges,
1 carton contain 12 boxes.

5 YEARS WARRANTY
2 years for applicators!



MANUFACTURED BY



H&O EQUIPMENTS

Reinventing Cryosurgery

USA Office

115 Fairchild St, Suite 370,
Charleston, SC 29492
USA

Phone : +1 888-248-2838
cryoprobe@ho-equipments.com

Operating hours
Monday – Friday
9:00–17:00 (GMT -4)

Head Office

Rue des Journaliers 1
7822 Ghislenghien
Belgium

Phone : +32 68 26 86 00
info@ho-equipments.com

Operating hours
Monday – Friday
9:00–17:00 (GMT +2)

www.ho-equipments.com
info@ho-equipments.com

DISTRIBUTED BY



laserméd
INNOVATING MEDICINE

www.laserméd.ch
info@laserméd.ch
+41 71 454 70 30